






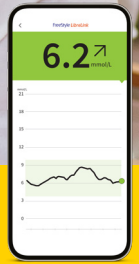


HOW TO GUIDE

Download the FreeStyle LibreLink app¹






FreeStyle LibreLink 



Apple

-  Go to App Store
-  Click Search
-  Type in FreeStyle LibreLink
-  Look for FreeStyle LibreLink – GB
-  Press **Get**
-  Install by entering your Apple ID²
-  Once downloaded open the app and follow the onscreen instructions to get set up

Android

-  Go to Play Store
-  In the search bar type FreeStyle LibreLink
-  Look for FreeStyle LibreLink – GB
-  Click **Install**
-  Once downloaded open the app and follow the onscreen instructions to get set up



Share your glucose data with your Healthcare Professional³

LibreView

Your Healthcare Professional's Practice ID

In the FreeStyle LibreLink app follow the steps below on your phone:

- 1 Menu
- 2 Connected Apps
- 3 LibreView **Manage**
- 4 **Connect to a practice**
- 5 Enter Practice ID from above **Next**
- 6 **Connect**
- 7 **Done**

You are now sharing your glucose data with your Healthcare Professional

FreeStyle Libre 2 

 **Abbott**
life. to the fullest.®

Images are for illustrative purposes only. Not actual patient or data.

1. The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle LibreLink requires registration with LibreView. 2. The mobile app is free to download and use. Internet connection is required to download the app, charges may apply. 3. The LibreView website is only compatible with certain operating systems and browsers. Please check www.LibreView.com for additional information. iPhone and App Store are trademarks of Apple Inc.

© 2022 Abbott. FreeStyle, Libre, and related brand marks are marks of Abbott. ADC-42037 v3.0 07/22.